

ROLAN EUROCLIPS

Clips Appliers & Titanium Atraumatic Haemostatic Clips

Euroclip - Clip Appliers



- precise loading of the clip
- firm grip of the clips
- safe closing
- color code
- full range of size



Large:

AI-321550	Clip Appliers, straight, 190 mm
AI-321552	Clip Appliers, right angled, 190 mm
AI-321554	Clip Appliers, straight, 280 mm
AI-321556	Clip Appliers, right angled, 280 mm



Medium Large:

AI-321540	Clip Appliers, straight, 190 mm
AI-321544	Clip Appliers, right angled, 190 mm
AI-321542	Clip Appliers, straight, 280 mm
AI-321546	Clip Appliers, right angled, 280 mm



Medium:

AI-321524	Clip Appliers, straight, 150 mm
AI-321526	Clip Appliers, straight, 190 mm
AI-321520	Clip Appliers, right angled, 190 mm
AI-321522	Clip Appliers, straight, 280 mm
AI-321528	Clip Appliers, right angled, 280 mm



Small:

AI-321530	Clip Appliers, straight, 150 mm
AI-321532	Clip Appliers, straight, 190 mm
AI-321534	Clip Appliers, right angled, 190 mm

Euroclip - Titanic Atraumatic Haemostatic Clips



easy and convenient

- soft loading system
- sticky tape at the back of the cartridge

safe and up to-date

- the chevron shape facilitates the safe „tip-first“ closing action
- serrated inner surface and a heart shaped cross-section of the clips allow secure closing
- solid titanium gives a scatter-free CT image and a MIR compatibility

sterile clips

- sterile packed clips in a peel pack



Large:

AI-74CL6	6	20	120
	Clips / Cartridge	Cartridge / Box	Clips / Box



Medium Large:

AI-74CML6	6	20	120
	Clips / Cartridge	Cartridge / Box	Clips / Box



Medium:

AI-74CM6	6	20	120
	Clips / Cartridge	Cartridge / Box	Clips / Box




Small:

AI-74CS6	6	20	120
	Clips / Cartridge	Cartridge / Box	Clips / Box

Euroclip - Clip Appliers

INSTRUCTION FOR USE

- Remove adhesive back on clip cartridge
- Place cartridge on stable surface
- Hold applicator between fingers as shown on the picture
- Introduce applicator jaws vertically into clip cartridge
- Push applicator fully into clip cartridge
- Remove applicator vertically to withdraw clip

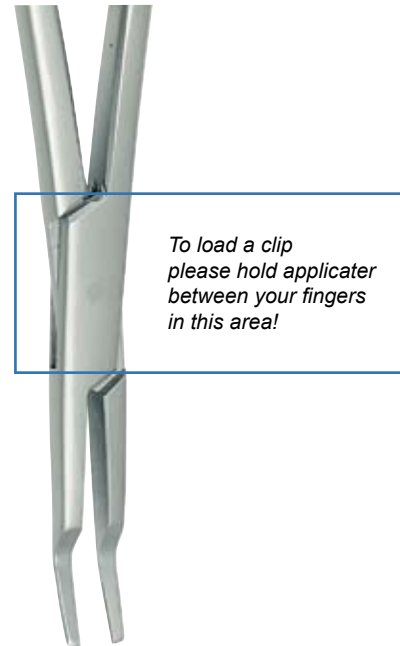

CLIP PLACEMENT

Place the loaded clip around the vessel or tissue to be occluded

Firmly squeeze the applicator handles together closing the clip

Release handles and remove applicator

Euroclip - Clip Appliers

SAFE AND UP-TO-DATE

- the chevron shape facilitates the safe “tips-first” closing action
- serrated inner surface and a heart shaped cross-section of the clips allow secure closing
- solid titanium gives a scatter-free CT image and a MIR compatibility



Schlangeneckstraße 16
79346 Endingen
Germany

Telefon: +49-(0) 7642 / 920 516
Telefax: +49-(0) 7642 / 920 517

e-mail: info@rolan-instruments.de